



Product Information

A.I.TAGS

Name:	A.I.TAGS	
Pack Sizes/Codes:	50/AHD AITAGS50 5000/AHD AITAGS5000	
Main Points:	<ul style="list-style-type: none">• A.I.TAGS use a unique Rub-Off technology that won't scratch off – resulting in no false positives.• No external glues or messy chalk to deal with. Simply peel off the A.I.TAGS Detector and apply.• The strong adhesive will ensure the detector stays in place – long after standing heat has passed.	
Status:	OTC	
Ingredients:		
Indications:	<ul style="list-style-type: none">• A.I.TAGS are an excellent aid for dairy and beef producers in detecting cows in mounting-heat. Apply the detectors to your cows prior to coming into heat, when synchronising estrus cycles or after breeding and embryo transplant to check for return heat.	
Action:	The detectors have a dark-grey, 'rub-off' surface with a bright coloured signal layer underneath (in red, fuchsia, green, or yellow). This bright colour, resulting from the rubbing pressure, begins to show through the grey layer after the first mounting. Additional mountings result in increased colour exposure.	
Administration:	Keep tags at body temperature prior to application Using a rubber brush, remove dust, dirt and loose hair Use a clean dry cotton glove or tack cloth to remove remaining particles prior to application Apply tags across spine between hip and pin bone. Press firmly with hand and rub onto hair securely	
Dosage:		
Special Precautions:	Best if not applied in wet conditions Do not use curry combs Do not trim hair, do not use oil or alcohol based products to clean the animal prior to application	
Storage:	Do not store in direct sunlight	
Registration:	Exempt	
Category:	Fertility support	



Animal Health Direct Limited

PO Box 8015 | Havelock North | 4157 | t: 06 873 3611
 e: sales@ahdltd.co.nz | ahdltd.co.nz

